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Recoverini™ Athena/Aphrodite

Version as of 07/2025

Thank you for choosing a quality product from Postheá.

Our compression bra is a medical device intended for postoperative care following breast surgery. Due to its material properties, it can also be worn as a functional bikini top for leisure activities.

This product is a Class I medical device in accordance with Regulation (EU) 2017/745 on medical devices (MDR) and bears the CE marking. It has been developed, manufactured, and tested in compliance with the applicable general safety and performance requirements.

2.1 Intended Purpose / Indications

The compression bra is designed for:

- Stabilizing operated breast tissue following surgical procedures (e.g., breast augmentation, reduction, mastectomy, reconstruction)
- Supporting wound healing through controlled compression
- Reducing postoperative swelling and edema
- Protecting scar tissue from friction or mechanical irritation
- Stabilizing implants or prostheses
- Additionally: Wearing during leisure or swimming activities thanks to UV protection and resistance to chlorine and saltwater

2.2 Contraindications / Risks

Relative contraindications (use only after consultation with a physician):

- Skin diseases in the application area
- Very sensitive or irritated scar tissue
- Radiodermatitis
- Material incompatibilities (e.g., elastane, polyamide)

2.3 Potential Side Effects

Incorrect sizing or improper use may cause pressure sensitivity, irritation, circulatory disturbances, or allergic reactions.

Should any undesirable side effects occur, discontinue use immediately and consult your physician.

2.4 Application Instructions

- The product is intended for direct postoperative use.
- The initial fitting should be performed by qualified medical personnel.
- The bra must not be applied directly onto open wounds.
- Depending on the surgical indication, the bra can be combined with a compression belt.

2.5 Determining the Correct Size

Measure the underbust circumference while standing.

The cup size is adaptable due to the material's elasticity and is suitable for Cups A–F (universal fit).

Underbust Circumference (cm)	Corresponding Bra Size	Size Designation
58–67	60/65	XS
68–77	70/75	S
78–87	80/85	M
88–97	90/95	L
98–107	100/105	XL

2.6 Putting on the Recoverini™

Put on the Recoverini™ by slipping it over your arms from behind so that the back panel rests against your back. Place the straps on your shoulders and fasten the front closure using the hook-and-eye fasteners. Check the fit and adjust the straps individually if necessary to ensure optimal support and comfort.

2.7 Material Composition

Main material:

- 79% polyamide, 21% elastane (Lycra® XTRA LIFE™), weight: 220 g/m²
- High resistance to chlorine and UV light, pilling resistance rating: 4/

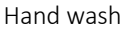
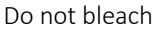
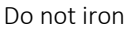
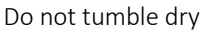
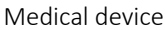
2.8 Warning Notice

Swimming or sunbathing while wearing the compression bra should be avoided for approximately the first 3–4 weeks after surgical procedures in the chest area. Clearance for such activities must be given exclusively by the attending medical professional.

2.9 Care Instructions

- Follow care label instructions
- Close all fastenings before washing
- Hand wash recommended
- Do not iron, do not bleach

2.10 Label Symbols

Symbol	Explanation
	Hand wash
	Do not bleach
	Do not iron
	Do not tumble dry
	Manufacturer
	Manufacturing date & batch number
	CE conformity marking
	Medical device
	Follow instructions for use
	Unique Device Identification (UDI)

2.11 Disposal

- Dispose of as household waste.
- The product is intended for multiple use by a single patient.

2.12 Reporting Obligations & Contact

In case of serious incidents, please contact your dealer or the manufacturer directly at: qc@posthea.com

2.13 Legal Notice

Postheá Inh. Tatjana Alimbakis
Bessemerstraße 51, 1st Floor
12103 Berlin, Germany